






**BiteMAX**



**METAL TRIPLE TRAY**



## **BITEMAX...** the multitasking tray

-  **TRIPLE FUNCTION-** Upper impression + lower impression + bite all together
-  **CURVED SHAPE-** to accommodate for arch profile
-  **IDEAL LENGTH AND WIDTH** for crown & bridge dentistry
-  **MEDICAL GRADE STAINLESS STEEL WITH ANODIZATION-** for the charming look
-  **AUTOCLAVABLE FRAME-** reusable and cost effective
-  **DISPOSABLE INSERTS-** 25 complimentary with tray
-  **REFILL INSERTS** available

For more details and videos visit [www.mikdental.in](http://www.mikdental.in)



## **INDICATIONS**

Impression of one-to-three posterior teeth on the same side.

## **CONTRAINDICATIONS**

Distal most tooth in the arch,  
Interference from third molar or tori to correct seating of the tray.

## **ADVANTAGES**

Faster than standard full-arch techniques,  
Much less material is used,  
No gag,  
More comfortable for patient,  
No invisible distortion owing to use of metal frame,  
No issues of cross-contamination owing to use of disposable sleeves.





## TECHNIQUE

**Step 1:** Try the tray in the patient's mouth to check if they can close correctly in MIP, making sure the distal-most tooth clears the posterior bar of the tray.

**Step 2:** Following the one step technique, dispense light body intra-orally and load the tray with putty on both side.

**Step 3:** Seat the tray intra-orally and ensure the patient closes into MIP.

**Step 4:** Once set, retrieve the tray with a snap.

Superior performance of the triple tray technique, combined with the high-quality materials used in its making, BiteMAX is all set to provides you with an ideal impression first time, every time. Time to give it a try!!!



**BiteMAX**

**INSERT REFILLS**

50 pieces in a bundle package







Follow us on  
*Instagram*



[WWW.MIKDENTAL.IN](http://WWW.MIKDENTAL.IN)

+91 9833400664/ +91 9111212912



Follow us on  
**Facebook**