







ISO  ATOR

**A GENIUS
QUADRANT
RETRACTOR**



ISOLATOR... the charismatic quadrant retractor

-  **DUAL FUNCTION-** Keeping the Mouth Propped Open + Tongue Deflection
-  **IDEAL DESIGN** keeping the patient extremely comfortable
-  **SET OF 10 PIECES-** 2 Bite Blocks and 8 Tongue Deflectors
-  **ADJUSTABLE INSERTS-** For progressive tongue Depression and/or Deflection
-  **VERSATILE-** Available in 2 sizes; Adult and Pedo
-  **AUTOCLAVABLE-** Reusable


For more details and videos visit www.mikdental.in

ISOLATOR... the charismatic quadrant retractor

INDICATIONS

ISOLATION of a quadrant to provide a saliva free environment for:

- Cavity Preparation,
- Composite Filling,
- Crown Cutting,
- Scaling, etc.

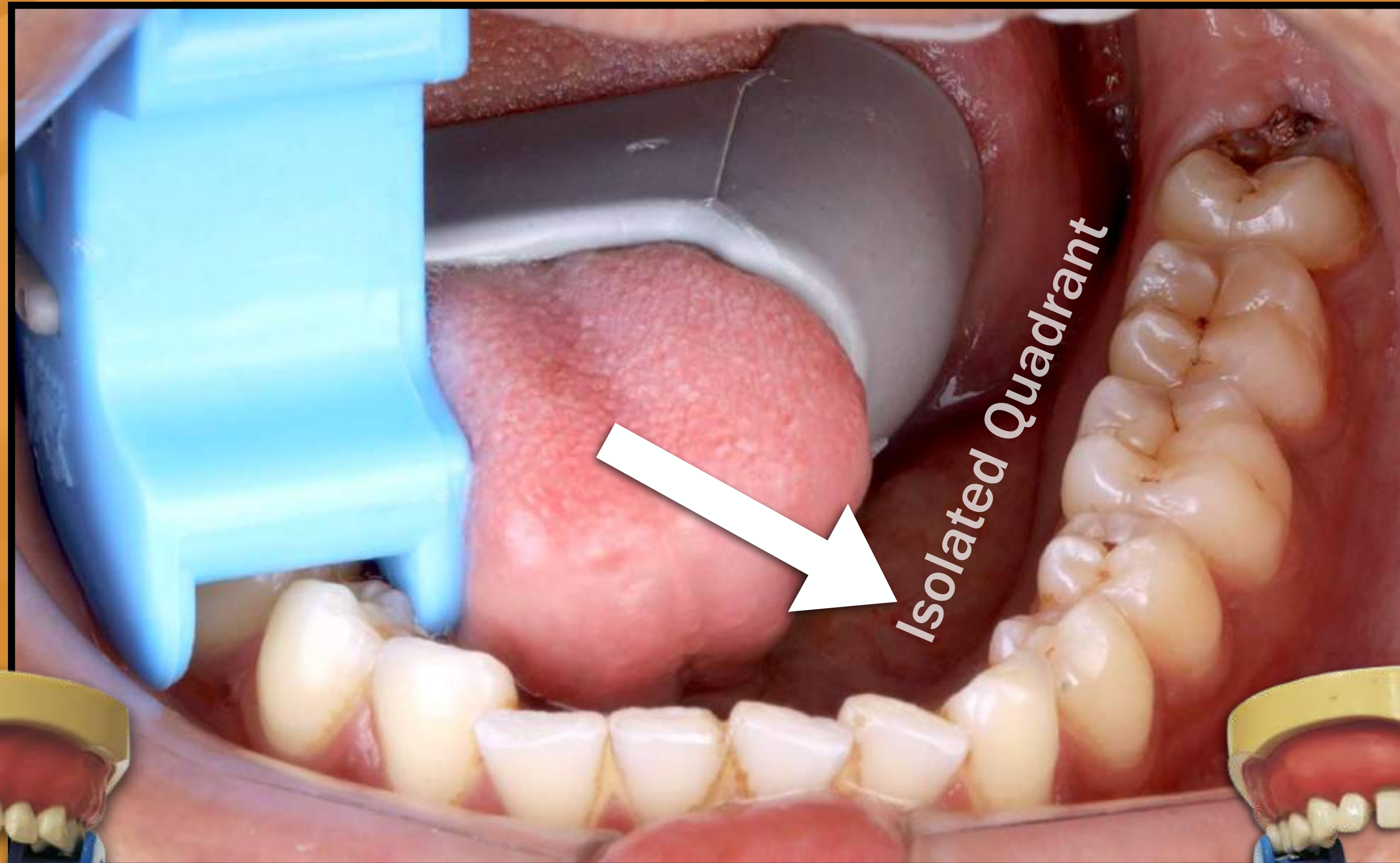

TECHNIQUE

Step 1: Outside the mouth, start with orienting the bite block such that the wider end points towards the front and the narrower end faces posteriorly.

Step 2: Then insert the tongue deflector into the appropriate slot of the bite block. If working on the right side of the oral cavity, choose the deflector with I and IV mentioned on the piece; and if working on the left side of the oral cavity, choose the retractor with II and III mentioned on the deflector.

Step 3: Place the assembly intraorally and have the patient bite down on the block and slide the tongue forward through the deflector.

Step 4: Place rolled cotton along with a suction tip in the retracted vestibule to aid isolation further.

SCAN TO WATCH VIDEO



25 Inserts Complimentary



Follow us on
Instagram



WWW.MIKDENTAL.IN

+91 9833400664/ +91 9111212912



Follow us on
Facebook