



# PanoraView



# **PANORAVIEW...** the GENTLE-TOUGH retractor



**WIDE VIEW-** Cheek Retractor **BEST SELLER PRODUCT**



**FIRM YET SOFT, COMFORTABLE & LONG LASTING**



**DESIGNED FOR PROGRESSIVE RETRACTION-** Unique design



Ideal for **LONG DENTAL PROCEDURES**



Ideal for **BITE REGISTRATION**



Ideal for **DENTAL PHOTOGRAPHY-** Clear with matt finish on handle



**AUTOCLAVABLE AND REUSABLE**



**PATENTED DESIGN**

For more details and videos visit [www.mikdental.in](http://www.mikdental.in)



# TECHNIQUE

## SEATING:

1. For long duration procedures, it helps to start with application of lip balm or petroleum jelly on the patient's lips prior to inserting the PanoraView cheek retractor.
2. Hold both ends of the retractor near the base of their handles and squeeze closer to each other.
3. Position the connector bow near the patient's chin and instruct the patient to open wide.
4. Gently insert the intraoral extension of one side and then the other followed by manipulating the patient's lip to seat comfortably along the curve of the retractor. The retractor is now ready to serve its purpose.

## RETRIEVING:

Once the procedure is completed, retrieve the device by holding the retractor handles and squeezing them closer (like before). Next, release the intraoral extension on one side followed by the other with the patient's mouth slightly open.

For more details and videos visit [www.mikdental.in](http://www.mikdental.in)





Follow us on  
*Instagram*



**WWW.MIKDENTAL.IN**

**+91 9833400664/ +91 9111212912**



Follow us on  
**Facebook**