

VENEER BURKAT

MK
DENTAL

**Precision
Crafted.
Perfect
Smiles.**

Specially Designed for
Porcelain Laminate
Veneers



ACCURATE
PREPARATION



PREMIUM
QUALITY










TIME
EFFICIENT



DURABLE
& RELIABLE

VENEER PREPARATION KIT... *the cosmetic assembly*

-  Concept by **DR. MOEZ I KHAKIANI**
-  Ideal for **PORCELAIN AND COMPOSITE LAMINATE & VENEER** preparations
-  **SMALL** yet **COMPLETE**
-  **5 BURS-** Gold series
-  **FASTER** CUTTING, **LONGER** LASTING **DIAMONDS**
-  **INNOVATIVE DEPTH CUTTING BURS-** for graduated preparation
-  **AUTOCLAVABLE CHOCOLATE** shaped kit
-  **PATENTED DESIGNS**

For more details and videos visit www.mikdental.in

VENEER BURKIT



PLV 0.3

This is a single tier, self-limiting depth gauge bur with a working diameter of **0.3 mm**. It has a non-cutting shank that extends above and below the working tip. This ensures that depth grooves of **0.3 mm** are created, regardless of the labial curvature of the tooth.



USE:

To scribe depth grooves on the cervical plane of the tooth where the enamel is the thinnest.



PRECISE
DEPTH CONTROL



SAFE
& RELIABLE



DIAMOND COATED
FOR LONG LIFE



PERFECT FOR
VENEERS

For more details and videos visit www.mikdental.in

VENEER BURKIT



PLV 0.5

SINGLE TIER • SELF-LIMITING DEPTH GAUGE BUR

This is a single tier, self-limiting depth gauge bur with a working diameter of **0.5 mm**. Works on the same principle as that described for the PLV 0.3 bur. It ensures the depth grooves achieved are **0.5 mm** deep, regardless of the labial curvature of the tooth.



USE:

To scribe depth grooves on the middle and incisal planes of the tooth where greater reduction is desired.



PRECISE
DEPTH CONTROL



SAFE
& RELIABLE



DIAMOND COATED
FOR LONG LIFE



PERFECT FOR
VENEERS

For more details and videos visit www.mikdental.in

VENEER BURKIT



DC 1.2F

Deep Chamfer Finishing Bur

Ø 1.2
mm

Tip Diameter: 1.2 mm

Type: Deep Chamfer Finishing

This is a **Deep Chamfer** Finishing bur with a tip diameter of **1.2 mm**. As this is a deep chamfer design, it is to be used up to half its depth to avoid formation of a lip of enamel at the margin.



USE

For '**VENEER**' preparation-
A wider bur is used as it needs more reduction than Laminate, as it is designed to change the underlying tooth shade.



Deep Chamfer
Design



Tip Diameter
1.2 mm



For Veneer
Preparation



Greater Control
& Tactile Sensation



Conservative
Restorations

For more details and videos visit www.mikdental.in

VENEER BURKIT



DC 0.8F

Deep Chamfer Finishing Bur

Ø 0.8
mm

Tip Diameter: **0.8 mm**

Type: **Deep Chamfer Finishing**

This is a **Deep Chamfer** Finishing bur with a tip diameter of **0.8 mm**. As this is a deep chamfer design, it is to be used up to half its depth to avoid formation of a lip of enamel at the margin.



For '**LAMINATE**' preparation as it needs less reduction.



Being a finishing diamond, it offers **greater control** and **tactile sensation** when preparing conservative restorations like porcelain laminates.



Deep Chamfer
Design



Tip Diameter
0.8 mm



For Laminate
Preparation



Greater Control
& Tactile Sensation



Conservative
Restorations



VENEER BURKIT



DC 1.2EF

Extra Fine Deep Chamfer Polishing Bur

Ø 1.2
mm

Tip Diameter: 1.2 mm

Type: Extra Fine Deep Chamfer Polishing

This is an **Extra Fine Deep Chamfer Polishing** bur with a tip diameter of **1.2 mm**.



To **polish** the veneer preparation.

Close adaptation between the Porcelain Laminate Veneer and the underlying tooth structure is extremely important for ensuring long term success with this treatment protocol. A smooth polished surface helps achieve this requirement.



Extra Fine
Polishing



Tip Diameter
1.2 mm



For Veneer
Preparation



Smooth
Polished Surface



Long Term
Success

For more details and videos visit www.mikdental.in



MIK

D E N T A L

PRECISION IN EVERY PREPARATION,
EXCELLENCE IN EVERY SMILE.



WWW.MIKDENTAL.IN



+91 9833400664